


West VIAL2BAG ADVANCED® ADMIXTURE DEVICES: Quick Reference Guide

The Vial2Bag Advanced® 13mm and 20mm admixture devices are indicated to serve as a connection between a 50mL, 100mL, or 250mL IV bag, vial with 13mm and 20mm closures, respectively, and an external IV administration set. The integrated vial adapter makes it possible to reconstitute and/or admix drugs prior to administration to the patient. Indicated for adolescent and adult patients only.

Consult Vial2Bag Advanced® 13mm and 20mm Admixture Device IFU for full product preparation and use detail



Scan to view full IFU and instructional video

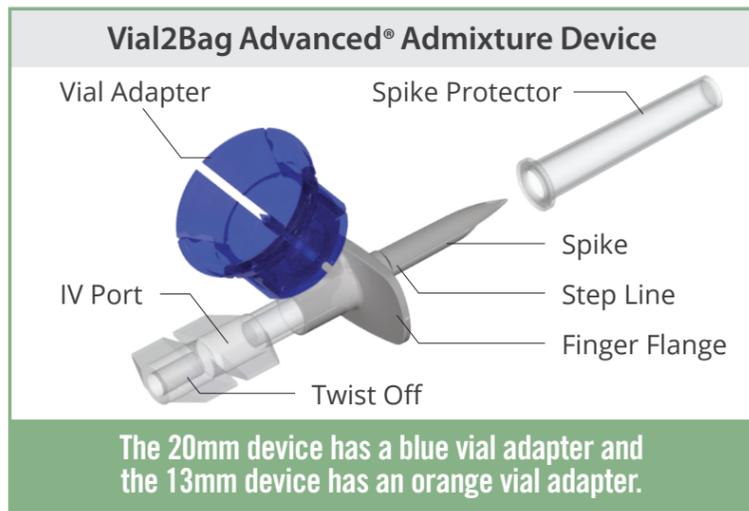


The usage steps illustrated here apply to both the Vial2Bag Advanced® 20mm and 13mm admixture devices, with the 20mm device serving as the representative example

1 ATTACH VIAL2BAG ADVANCED® ADMIXTURE DEVICE TO VIAL

- Place vial on a stable, flat surface
- Hold Vial2Bag Advanced® admixture device by the finger flange
- Push Vial2Bag Advanced® admixture device into center of vial until it is attached




- Remove Vial2Bag Advanced® admixture device spike protector
- Spike Vial2Bag Advanced® admixture device into IV bag until it is up to the step line

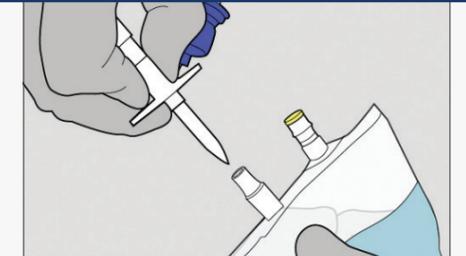


2 DISSOLVE OR DILUTE DRUG

- Hold bag with vial down
- Fill vial until half full by squeezing bag repeatedly

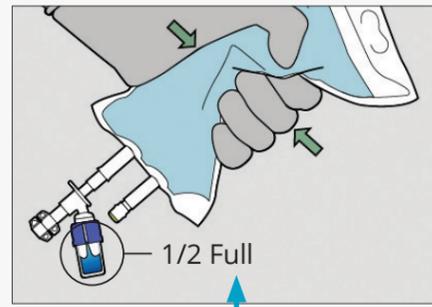


- Fold the bag behind the vial
- Hold vial and bag together
- Admix per drug manufacturer's package insert

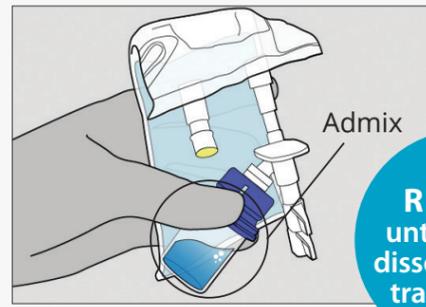


REPEAT
until drug is dissolved and transferred

- Hold bag with vial up
- Empty vial by squeezing bag to force air into vial

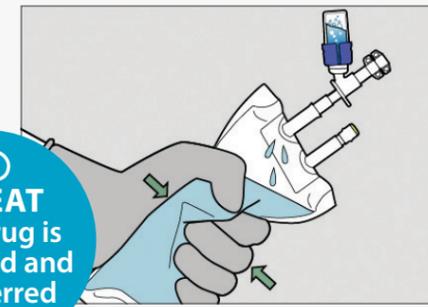

- Inspect vial for any remaining drug
- Repeat until drug is dissolved and transferred

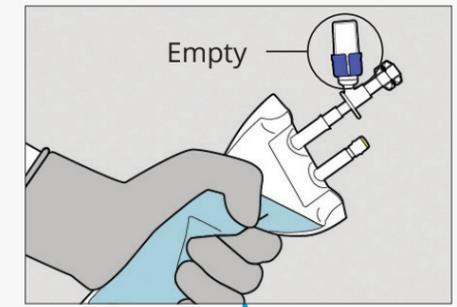


3 CONNECT IV SET TO VIAL2BAG ADVANCED® ADMIXTURE DEVICE

- Twist off end to open IV Port
- Remove protector from IV set spike



- Spike the IV set per its instructions to the Vial2Bag Advanced® admixture device IV port until it is tightly secured
- Empty drug from vial, if needed by repeating Step 2.
- Leave vial in place
- Do not rotate vial

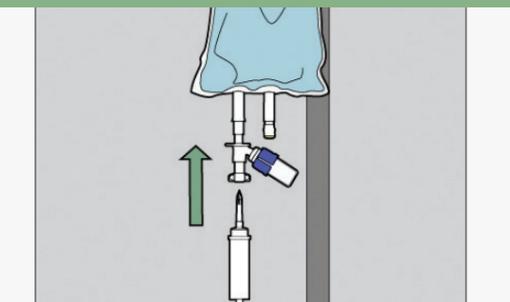

Rx Only

Exclusively Distributed by: Progressive Medical, Inc.

997 Horan Dr. • Fenton, MO 63026

Ordering Information: Phone: 314-961-5786

Fax: 314-961-4535 • cs@progressivemedinc.com

STERILE EO Do not use if package is open or damaged Single use only

Vial2Bag Advanced® 20mm admixture device contains or presence of phthalate: bis (2 - ethylhexyl) phthalate (DEHP)

Vial2Bag Advanced® 20mm admixture device: Ref. No.: 1090209 Rev 3.0
Vial2Bag Advanced® 13mm admixture device: Ref. No.: 1090270 Rev. 1.0

Do not attach a Vial2Bag Advanced® admixture device to another Vial2Bag Advanced® admixture device.

The Vial2Bag Advanced® 13mm and 20mm admixture devices are 510(k) cleared by the United States Food and Drug Administration (FDA). The use of the Vial2Bag Advanced® 13mm and 20mm admixture devices should not be interpreted as modifying, extending, or superseding a drug manufacturer's labeling recommendations for storage and expiration dating, unless otherwise limited by USP <797> compounding standards. Refer to drug manufacturer's labeling and use instructions for recommendations, USP <797>, and applicable institution policy for shelf life and sterility information of reconstituted product and admixture device compatibility. Compatibility of the Vial2Bag Advanced® 13mm and 20mm admixture devices with all drug products has not been confirmed. Do not use the Vial2Bag Advanced® 13mm and 20mm admixture devices with lipids. Failure to follow the instructions provided may result in inadequate medication reconstitution, dilution, and/or transfer, possibly leading to overdose or underdose and/or delay in therapy. Products shown are for INFORMATION purposes only and may not be approved for marketing in specific regions. Please contact your West Pharmaceutical Services, Inc. (West) representative for product availability. Vial2Bag Advanced and the Blue Vial Adapter are trademarks or registered trademarks of West Pharma. Services IL, Ltd., a subsidiary of West Pharmaceutical Services, Inc. Important product and safety information and warnings at: <https://www.westpharma.com/-/media/WestPharma/Files/Products/1090209-InDesign-IFU-for-V2B-Advanced-20mm-22-08-2023-rev3.pdf> and <https://www.westpharma.com/-/media/WestPharma/Files/Products/IFU-1090270-V2B-13mm-IFU-28-11-2023-rev10.pdf>.