

Brainchild Surgical Devices, LLC


Lapro-Shark™

Laparoscopic Fascial Port Closure Device

SAFE

SIMPLE

PRECISE

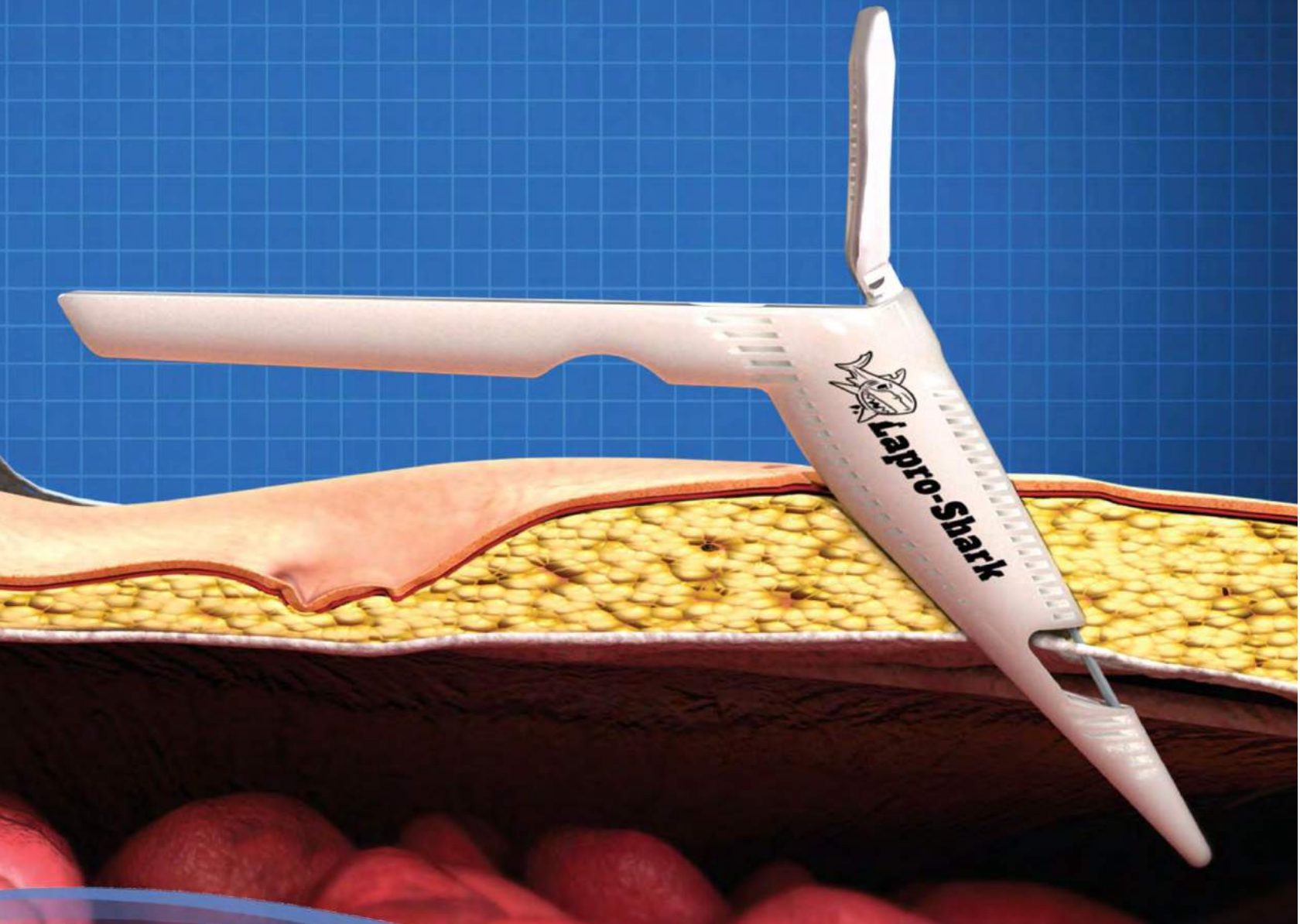

www.lapros shark.com



EXCLUSIVELY DISTRIBUTED BY:



Progressive Medical, Inc.

997 Horan Drive · Fenton, MO 63026

Phone: 800-969-6331

Email: CS@ProgressiveMedInc.com

Key Features and Benefits

SAFE

Protected needle: While other closure devices require free needle puncture into the abdominal cavity, the Lapro-Shark™ needle is always protected within the Lapro-Shark™ guide, thus eliminating the risk of injuring the intra-abdominal viscera.

SIMPLE

Automatic suture capture: Unlike other devices which require active suture retrieval in mid-air, the Lapro-shark™ is a remarkably simple device which automatically captures the suture, making port closure effortless, even for the novice.

PRECISE

Modified shape: While all other closure device guides are round, the Lapro-Shark™ has an elliptical body and notched tip. The novel shape of the Lapro-Shark™, is designed to better retract the skin and subcutaneous tissue while yielding improved and reproducible fascial capture.

Ergonomic Handle: The Lapro-Shark™ is the only closure device with a built in handle. The Lapro-Shark's™ ergonomic handle allows for easy manipulation and precise control over the depth of the fascial bite, regardless of abdominal wall thickness.

Instructions For Use



Insert Lapro-Shark™ and Capture Fascia



Insert Needle via Needle Guide



Insert Suture via Suture Guide



Withdraw Needle and Suture



Rotate Lapro-Shark™ 180° Capture Fascia of Opposite Side



Insert Needle via Needle Guide



Insert Other End of Suture via Suture Guide



Extract Needle and Device and Tie Suture in Usual Fashion

For more information please visit www.lapros shark.com

ORDERING INFORMATION

Item #	Description	QTY / Box
P10176	Lapro-Shark™ Fascial Closure Device	10

EXCLUSIVELY DISTRIBUTED BY:



997 Horan Drive, Fenton, MO 63026 · Phone: 800-969-6331 · Email: CS@ProgressiveMedInc.com

PMIOR0020revSEP16